EL RO

# MOBIAK®

**MEDICAL • GASES • FIREFIGHTING** est. 1977

Manual de utilizare:

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MEDICAL - GASES • FIREFIGHTING est. 1977

## PRODUCT DESCRIPTION

Anti-decubitus mattresses are specifically designed to treat and help bedsores in bedridden patients. They can be used for hospital or home use.

Evenly distribute the pressure on the body to prevent edema.

The pump switches the air pressure in the mattress, creating a sense of light massage and activate the immobilized due sprawling cells.

This product is indicated and suggested for patients who are likely to spend more than 15 hours a day in bed and for patients with low to medium risk of developing ulcers & bedsores (Stage I to II)

## OPERATION USE

### **For the Tubular mattress (0806286)**

- Place the air mattress over the bed mattress, having first stretch the bottom sheet, making sure the air inlet is at the side of the legs.
- Secure it with fixing straps below the mattress
- Place over the air mattress a thin breathable fabric, if not accompanied by its own cover, so it does not come in direct contact with the patient's skin.

### **For the pump (0806278)**

- Attach the mattress to the pump using appropriate hoses that are in the package, when you hear a "click", the connection will be secured. Connect the pump to the power supply, press the "ON/OFF" switch and turn the knob to the maximum in order to inflate the mattress, BEFORE been used by the patient
- When the mattress is fully inflated you can adjust the pressure range, depending of the user's requirements, using the switch.
- ATTENTION: the STATIC mode is for 20 minutes only!!!

## OPERATION USE

### **For the pump (0806278) :**

- ❑ Press the “orange” switch to select Static Pressure. This switch activates or deactivates the alternating air flow from the pump to the mattress and gives continuous flow.
- ❖ When there is an emergency to perform CPR on the patient, quickly pull the CPR valves to release air from mattress. The CPR valve are located at the head-end, left-hand-side of the mattress. The quick connector found on the pump unit can be disconnected for even faster deflation.
- ❖ There are four tubes, with ventilation holes on their edges, from the tubes of the anti-decubitus mattress

## EC Declaration of Conformity

**We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.**

# SPECIFICATIONS

<b>PUMP</b>	Voltage (V) - Frequency (Hz)	<b>AC220V/ 50HZ</b>
	Air Output	<b>≥ 5-6 lit/min</b>
	Pressure Range	<b>50mmHg – 110mmHg</b>
	Cycle Time	<b>12 min</b>
	Power Consumption	<b>≤ 7 W ± 10%</b>
	Noise	<b>≤ 25 dB ± 10%</b>
	Dimensions (LxWxH)	<b>26 x 12,5 x 8,5 cm</b>
	Pump Weight	<b>1,3 Kg</b>
<b>STRIPED MATTRESS</b>	Max Load	<b>145 Kg</b>
	Color	<b>Blue</b>
	Material	<b>Medical Grade PVC</b>
	PVC Thickness	<b>0,45 mm</b>
	Dimensions (LxWxH)	<b>190 x 85 x 11,5 cm (17 cells)</b>
	Hose Length .	<b>150cm</b>
	Weight	<b>5 Kg</b>

# MAINTENANCE

- To clean the mattress Use a soft cloth or sponge & water with mild soap. ATTENTION: Do not use acidic & caustic dilutions.
- Do not smoke & do not use any flame near the mattress.
- Do not have the mattress in contact with direct heat sources.
- If you need to save the mattress, empty of air, folding it
- Keep or store the pump in a clean, dry and well ventilated environment.
- Before cleaning the pump, disconnect from the electrical supply.
- Clean the pump with a dry cloth. ATTENTION: Do not bring the pump into contact with liquids.

## IMPORTANT (!)

### CAUTIONS

- Do not disconnect the hoses while the pump is running.
- Do not use sharp objects near the mattress.
- Do not exceed the maximum load (kg) recommended by the manufacturer.
- Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
- Do not use abrasive alcohol-based cleaning when the mattress. This could damage the material of the layer.
- Clean the upholstery after each daily use.

We recommend that you clean the upholstery with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.

- Protect the layer from scratches, cuts and punctures.
- Do not leave the product run on, If not using it

## IMPORTANT (!)

**In case of damage or malfunction of the pump, please contact your authorized and trained dealer.**

Do not use the product for a purpose not indicated in this manual

Only qualified personnel or trained for the treatment and prevention of bedsores, can operate with the device

MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

MOBIAC S.A reserves the right to change the information contained in this document without previous notice.

### ENVIRONMENTAL PROTECTION

 If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards

### SPARE PARTS

REF. NUMBER	DESCRIPTION
0806286	TUBULAR MATTRESS WITH VENTILATION & CPR
0806278	PUMP "PREMIUM"

The product has warranty for two (2) years in pump and six (6) months on the mattress and the other non-mechanical parts. The warranty covers factory defects and not damages resulting from improper use / maintenance of the product.

The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

## Warranty Form

### Buyers Data

Name: .....

Date of purchase: ..... S/N.....

LOT:.....Phone: .....

### Resellers Data

Name :  
.....  
.....

Address:.....  
.....

Phone: .....

**Reseller Stamp**

### INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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